


HANAU™ EJECTOR for HANAU™ EJECTOR TYPE FLASKS

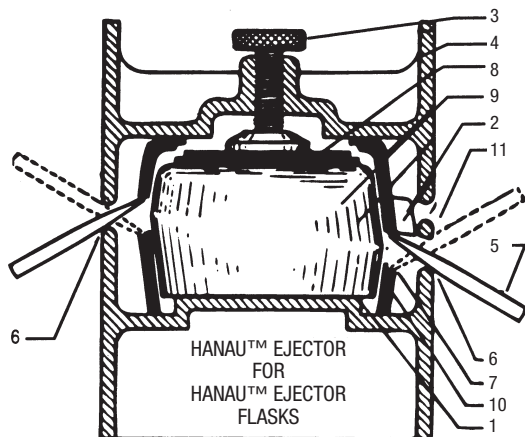

1. Base with raised portion fitting contour of Flask.
2. Heels of Flask aligned with frame of Ejector.
3. Thumb knob.
4. Removable disc in bottom section of Flask.
5. Pry bars.
6. Lower opening in frame serving as fulcrum for **Varsity Lower** Flask.
7. Slots in front and back of Flask.
8. Bottom section of Flask.
9. Invested case.
10. Middle section of Flask.
11. Upper opening in frame serving as fulcrum for **Varsity Lower** Flask.

AFTER REMOVAL OF FLASK COVER, place Flask with the open end upon the raised portion of base (1); the contour of this base coincides with that of the Flask and ready registration is simple by aligning the Flask with the base of the Ejector. The heels (2) of the Flask touch the upright of the Ejector.

Thumb knob (3) is screwed down by hand to contact disc (4) in the bottom section of Flask (8). No force whatsoever is necessary.

The flattened ends of pry-bars (5) are inserted in the openings (6) on both sides of Ejector to engage the slots (7) of the Varsity **LOWER** Flask – or into openings (6) and (11) for Varsity **UPPER** Flask.

Both pry-bars are simultaneously pressed down with equal pressure, and thus the bottom section (8) of the Flask is lifted from the investment (9).

The pry-bars are lifted upwardly while their flattened ends engage the slots (7) of the Flask and the middle section (10) is released from the investment.

Release knob (3), remove the Flask from Ejector and lift Flask sections from the investment.

If the Ejector is not fastened to bench, press pry-bars downwardly to release bottom of Flask. Turn Ejector upon its legs and press pry-bars down to free the middle section of Flask from investment.

The HANAU™ EJECTOR is furnished with two pry-bars and four screws for fastening to bench.

For use only with **HANAU™ EJECTOR TYPE FLASKS**.



RESTORATIVE ORAL HEALTH

Whip Mix Corporation - West

1730 East Prospect Rd., Suite 101 • Fort Collins, CO 80525

Toll-Free: 1-800-201-7286 • Fax: 1-970-472-1793 • www.whipmix.com

Whip Mix® and logos are registered trademarks and Hanau™ is a trademark of Whip Mix Corporation.

©2008 Whip Mix Corporation.