

On going Qualification of Whip Mix 3D Print Resins with Post Curing Units

	Verimodel OS - White	Verimodel OS - Ivory	Verimodel OS -Golden Brown	VeriGuide OS	VeriSplint OS	VeriCast	VeriTray	Dentca
G171/Otoflash	2X 2000 flashes	2X 2000 flashes	2X 2000 flashes	6000 flashes	6000 flashes	1 x 750 flashes	2X 2000 flashes	Pending
IntelliRay 600W/Uvitron	pending	pending	pending	5 min at the lowest settings of the rack	5 min at the lowest settings of the rack	pending	pending	10 minutes per side @ 50% intensity in Glycerin bath
SurnRay 400W /Uvitron	pending	pending	pending	5 min at the lowest settings of the rack	5 min at the lowest settings of the rack	pending	pending	10 minutes per side in Glycerin bath