


HANAU™
MODEL *Mate™*
PLASTERLESS ARTICULATOR



Instructions for Use



Overview of Hanau™ Model Mate™ Articulator

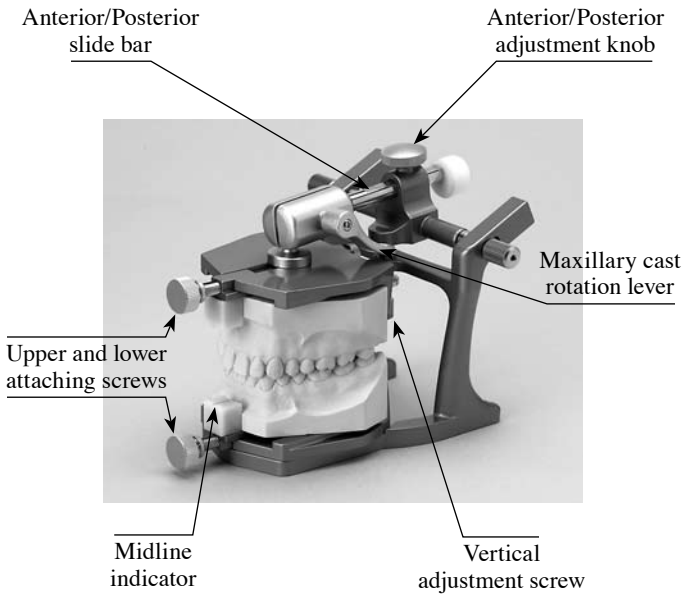

Instructions

- A. Casts of the upper and lower dentition are prepared in your customary manner.
- B. Secure the lower model first. Loosen the lower attaching screw, insert the model, lining up the midline indicator with the dental midline, and tighten the screw.
- C. Follow the above instructions for the upper model.
- D. Loosen the anterior/posterior adjustment knob and flip the lever forward. This will allow free movement of the upper model in all directions.
- E. Position the upper model into the lower model (centric occlusion) and lock the anterior/posterior adjustment knob making sure the flat side of the slide bar is parallel to the floor*. Close the lever by pushing it backwards.
- F. Turn the vertical adjustment screw (found in the rear of the articulator) to raise or lower the vertical dimension of occlusion. Once the casts are in proper position, lock into place by rotating the lock washer until contact is made with the articulator and secure in place.

* Once the model is locked, if the cast is able to move sideways the anterior/posterior slide bar is not in its correct parallel location.

Warranty

Whip Mix Corporation warrants this articulator to be free from defects in material and/or workmanship for a period of six months. In the event of a defect, please notify the factory in writing of the defect prior to returning the instrument.

Whip Mix Corporation will, at its option, either repair, replace, or issue credit for such defects. Because Whip Mix Corporation is continually advancing the design of its products and manufacturing methods, it reserves the right to improve, modify or discontinue products at any time, or to change specifications or prices without notice and without incurring obligation.



RESTORATIVE ORAL HEALTH

Whip Mix Corporation- West
1730 East Prospect Rd., Suite 101
Fort Collins, CO 80525
Toll-Free: 1-800-201-7286
Fax: 1-970-472-1793
www.whipmix.com

Whip Mix® and logos are registered trademarks and Hanau™ and Model Mate™ are trademarks of Whip Mix Corporation