# Hanau™ Springbow

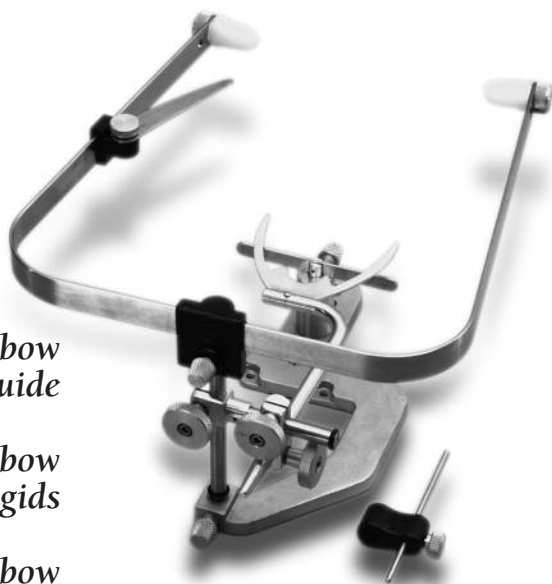
*Hanau™ Springbow  
Quick Reference Guide*

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

RESTORATIVE ORAL HEALTH

# The Self Centering Hanau™ Springbow/Technique for Use

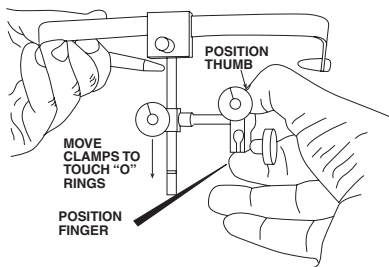
DO NOT tighten a clamp without first inserting a bitefork stem into the clamp. Such action will distort the clamp and render it inoperable or cause the bitefork stem to bind. Before each use lubricate the bitefork stem and the rods on which the clamps glide. Note: The Springbow may be made sterile via steam autoclave. Place the bitefork with registration material in the patient's mouth.

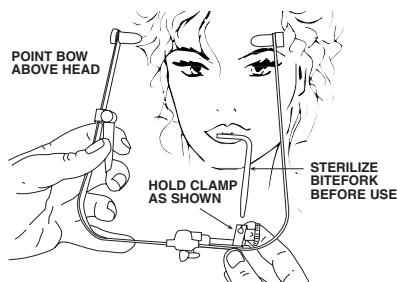
1. Unlock thumbscrews and slide clamps to touch "O" rings.

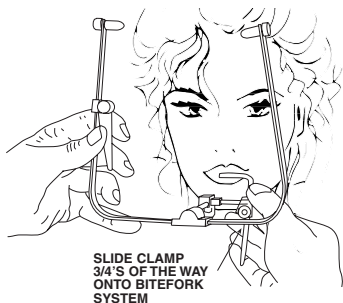
2. Hold clamp in the right hand; the left hand holds the Springbow (see drawing). Tilt bow, earpieces above patient's head.

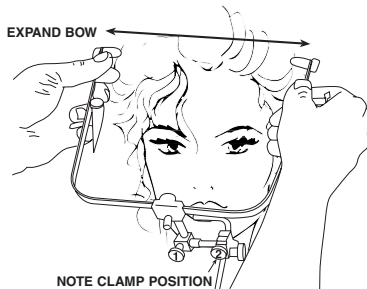
3. Guide clamp onto bitefork stem and 3/4's of the distance toward patient.

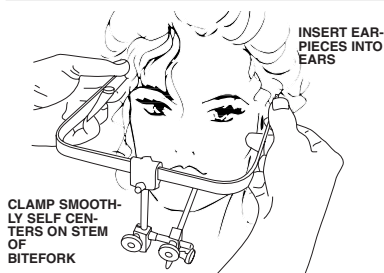
4. Position your thumbs inside the bow and near the earpieces.

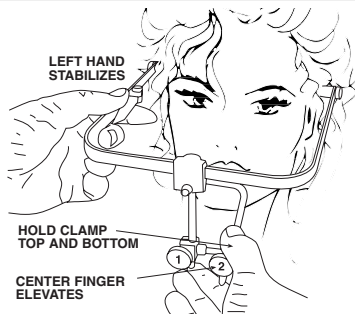
5. In one smooth motion expand the bow and seat the earpieces into the patient's ear canals. (The clamp on the bitefork stem self-centers the correct front to back distance for this patient.)

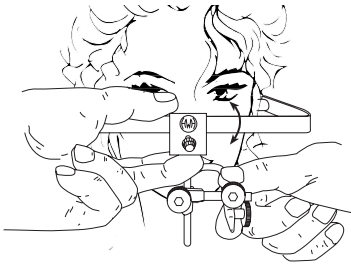
6. Place the middle finger under the vertical rod, thumb and index finger supports clamp.

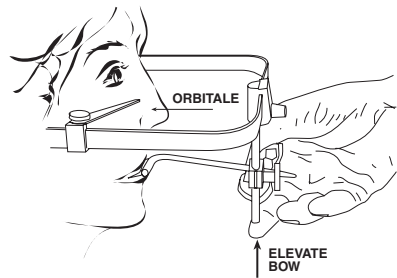
7. Technique hint to achieve smooth movement “Simultaneously push Springbow downward with left hand and lift clamp upward with right hand”.

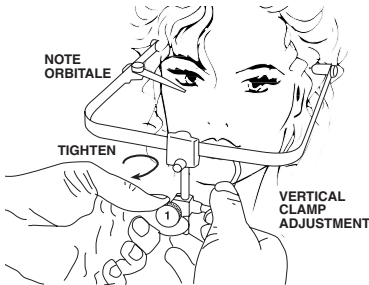
8. Middle finger elevates the pointer and the bow until the pointer is parallel to the orbitale.

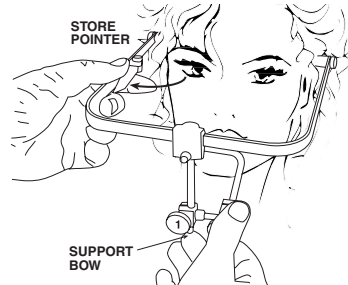
9. Tighten the vertical clamp 1.

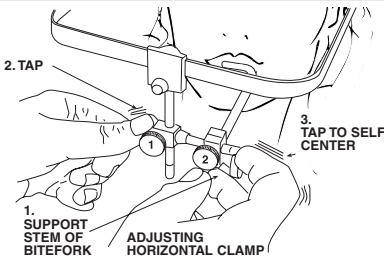
10. Move the pointer to the storage (neutral) position.

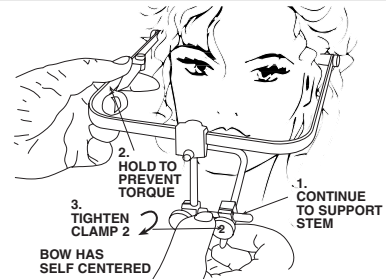
11. Support the stem of the bitefork with your middle finger. Lightly tap the right and left side of the horizontal rod (note: the bow self centers on the patient).

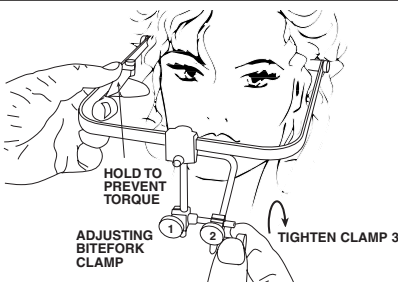12. Hold the bow at the front left corner to prevent torquing, tighten horizontal clamp 2.

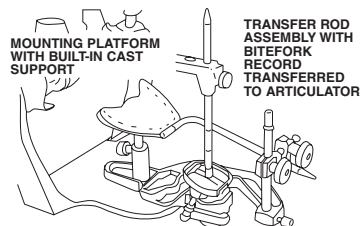13. Tighten clamp 3 which locks the stem of the bitefork.

14. Remove the facebow from the patient. Again, check the thumbscrews for tightness. Send this record with centric, right and left interocclusal records and impressions to the laboratory.

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©2008 Whip Mix Corporation
Whip Mix® and Hanau logo are registered trademarks and Hanau™ is a trademark of Whip Mix Corporation.
Whip Mix Corporation - West
Whip Mix Europe GmbH