

# Denar® FlexTrack from Whip Mix®

## Articulator Accessory Instructions



The FlexTrack provides the flexibility of movement with both open and closed condylar tracking systems. For use on Denar Mark 310 and Mark 320 articulators, only.

## The Most Flexible Articulator System: Denar® Mark 310 and Mark 320

### CLOSED TRACKING FOSSAE

Delivers consistently accurate movements by keeping the condyles in contact with the fossae at all times during functional movements. Perfect when making guide tables or adjusting excursive movements since the upper member cannot lift away from the lower member condyle, maintaining accurate programming. This will insure the most accurate adjustments of your restorations possible on a semi-adjustable articulator.

### OPEN FOSSAE

Traditional "Open Fossae" design allows for diagnostic analysis and planning by permitting separation of condyles and fossae during function. This makes it possible to identify and diagnose occlusal interferences and other pathologic issues for the patient. FlexTrack enables you to have both systems in one articulator.

**Note:** FlexTrack does not fit the Denar® Mark 330 articulator.

## Installation

Remove the upper member by releasing the centric latch and elastic retainers from the shoulder screws.



Figure 1



Figure 2



Figure 3

First distinguish between right (R) and left (L) identified on the side of the flex track. The opening of the FlexTrack faces the operator (Figure 1). Then slide FlexTrack toward the center of the upper member with "R" & "L" exposed on the outside or follow the arrow. (Figure 2) After installing FlexTrack on both fossae, put the upper member back onto the articulator and add the elastic retainers. Your Denar Mark 310 or 320 is now ready to be used as a tracking system.

**Note:** You may need to rotate and separate the Flex Track slightly with your thumb to ease the track until it 'clicks' into place. There is a small tab on the lower inside part of each FlexTrack. The tab fits into a screw hole on the fossa, securing it in position.

## Removal

Remove the upper member. (Figure 3) Pressing with your thumb or finger inside the FlexTrack will release the tab, allowing the FlexTrack to slide off the fossae.

CHECK OUT THE  
VIDEO TUTORIAL



ROHL1001 Rev. B 05/26



Whip Mix Corporation 361 Farmington Avenue Louisville, KY USA 40209  
Phone: 502-637-1451 | [www.whipmix.com](http://www.whipmix.com)

© 2012 Whip Mix Corporation FN 20001047-F AA 1211